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Instruction:

Narmak medical products can be used as hot and cold compress. For using hot compress you must put it in a cotton bag and warm it by devices like microwave, toaster, radiator, heater and oven. It's better to do it several times, and per time warm it for 1 minutes by microwave, or put the bag in a refractory container on the low flame stove. For using cold compress put it in a plastic bag and keep it for a few minutes in refrigerator or freezer.

The Uses:

Narmak products relieve pain and muscle spasms. They are used to treat aches and pains as well as problems with joints, muscles and bones. These products also help to relieve pain caused by rheumatoid arthritis, sciatica, migraine, sinusitis, burn, contusion, kidney stone, colds, flu and covid (coronavirus disease). These products are also suitable for relieving menstruation and pregnancy pain, hemorrhoids, flatulence and etc. Narmak medical packs are useful to reduce stress and for relaxation and massage therapy.

Tips for using the product mentioned:

- * before placing the bags on the skin, make sure that the temperature is adjusted. This is especially important when designing a baby.
- * If the material inside the bag sticks together, open the material with the pressure of your fingers.
- * Use only the fabric in the product as a bag cover. Because the special washable cover of the product is made of porous fabric and will transfer the properties of the material to the body, so place the coated bag directly on the skin of the body and do not use other fabrics and clothes.
- * Avoid spilling water on the salt bag.
- * When using the product as a warm compress, wrap the bag in a cotton cloth and warm it to prevent it from getting dirty. When used as a cold compress, place the bag inside the plastic bag without a lid and place it in the refrigerator or freezer. And when placing the bag on the body, be sure to use the special cover of the product.
- * To prevent combustion, close the straps when heating if there are straps in the product.
- * The longer the bag is heated or cooled, the later it will lose its heat.
- * And for utilization of healing properties of material, if you have access to an electric mattress, you can put the bag on it the skin of body can be placed on the Narmak bag without the intermediary of clothes.

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NARMAK MEDICAL PRODUCTS Relaxing seeds

Treatment bags is included by salt and organic seeds

The basis of the idea in this product is based on salt therapy and thermotherapy (heat and cold therapy) This product is completely organic and without any chemical and unhealthy material

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